

Cleaning and Disinfection of the Dialysis Station Procedural Checklist #4

- Don gown, gloves, and impermeable mask/eye protection or face shield
- Remove all bloodlines and disposable equipment and discard in biohazardous waste; reprocess dialyzer, with all ports capped; transport dialyzer, bloodlines, etc. in a manner that prevents contamination of other surfaces; remove gloves
- Hand hygiene
- Don clean gloves; obtain EPA-registered disinfectant; use tuberculocidal disinfectant if blood is visible
- Wipe machine top, front, and side surfaces and dialysate hoses wet with disinfectant per manufacturer directions for use; if blood is visible, do second application of tuberculocidal disinfectant
- Empty prime waste receptacle: wipe all internal and external surfaces wet with disinfectant per manufacturer directions for use
- When chair is vacated, remove and discard all disposable supplies
- Fully recline chair and clean with disinfectant; wipe all external front-facing and side chair surfaces wet with disinfectant per manufacturer directions for use, including down sides of seat cushion and side tables
- Wipe all nondisposable items with disinfectant, including blood pressure cuff, TV controls, call button, data entry station, and counters around station
- If clamps are used, clean off visible blood and dirt and disinfect
- Discard cloth/wipe; remove gloves
- Hand hygiene

Note: Allow disinfectant contact time per manufacturer's recommendations for all checklist items. In other health care settings, patients vacate treatment area before cleaning and disinfection. This practice should be considered for dialysis facilities.